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Face Shield Instruction Manual

Description:

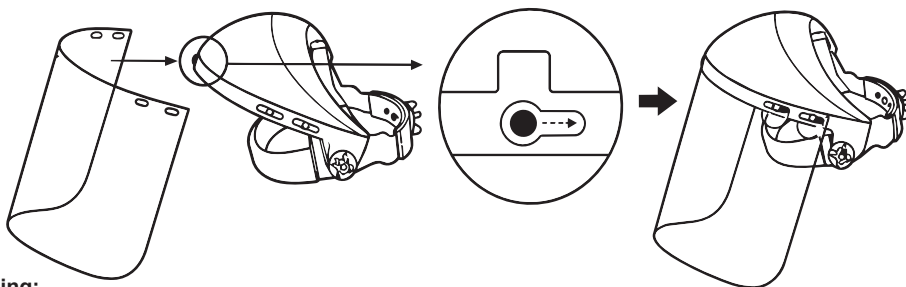
The face shields are designed for face and eye protection against basic impact level. They are light by weight and comfortable to wear. Additional eye protection such as spectacles or Goggles must be worn at all times when using these face shields. The materials used these face shield do not contain any substances known to cause allergic reactions. However, it may be possible for certain susceptible individuals to develop allergies to the material used.

Standards:

These protective face shields comply with the Regulation through the harmonized European standard ISO 16321-1:2022.

Fitting Visor:

Sliding the visor slots into the swivel posts, turning the swivel posts to hold the visors firmly.

Warning:

Faceshields only provide limited protection and are not unbreakable. Large objects and/or objects at high speeds may break or penetrate the faceshield and injure the wearer's face or eyes. Eye protectors against high speed particles worn over standard ophthalmic spectacles may transmit impacts, thus creating a hazard to the wearer. Any damaged faceshield must be replaced immediately. If protection against high speed particles worn over standard ophthalmic spectacles may transmit impacts, thus creating a hazard to the wearer. Protectors that have been subject to impact shall not be used and shall be discarded and replaced.

Specifications:

Material lenses: polycarbonate. Luminous transmittance: 90%

Marking on the shield:

EN Standard — 16321-1:2022 VAM E 1  Identification of the manufacturer — CE Marking

Compatibility of Marking:

If the impact level symbols are not equal on both the lens/filter and the frame, then it is the lower level that shall be assigned to the complete protector. The protections corresponding to the code numbers/letter 7, 9, CH are provided by the complete protector only if the respective symbols are equal on both the lens and the frame.

USAGE, STORAGE, AND MAINTENANCE:

In use, ensure the face shield fits comfortably over the face. This eye protector should be stored in a case, out of direct sunlight, away from chemicals and abrasive substances, in a clean, dry, dust-free environment when not in use or during transportation.

To clean, rinse in 1% solution of mild detergent and dry with a dry fluff-free soft cloth. DO NOT USE abrasive cleansers. If necessary, a non-aggressive household disinfectant may be used.

Service life:

The period of obsolescence of this product is 3 years.

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MED Consulting S.r.l.